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Lieferumfang:
576670 Duplikierküvette inkl.
6 Blockpins (561451),
5 feuerfeste Pins (576480)

Zubehör
576480 Feuerfeste Pins (Keramik) 25 er
576490 Doppelkonuspinn (Alu) 250 er
561451 Blockpins 100 er

USER MANUAL

Packing list:
576670 Duplicating flask incl.
6 Blockpins (561451),
5 Refractory pins (576480)

Accessories
576480 Refractory pins 25 pcs
576490 Tapered double ended pins (Alu) 250 pcs
561451 Blockpins 100 pcs.

MODE D'EMPLOI

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Trim the segments that are to be duplicated to a tapered form; this allows easier later removal from the silicone mould.

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Remove all parts except those dies that are to be duplicated.

Fit the Giroform base plate to the duplicating flask and close the swivelling fasteners on the underside.

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Adjust the partitioning bar. Place two pins outside the duplicating area (Blockpin - Art. No. 561451), move the partitioning bar into contact with the pin and fasten the locking screw.

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Pour the duplicating silicone into the flask and allow to set according to the manufacturer's instructions.

CAUTION: Use silicone with low Shore hardness (approx. 20), as this will ensure easier removal of the dies.

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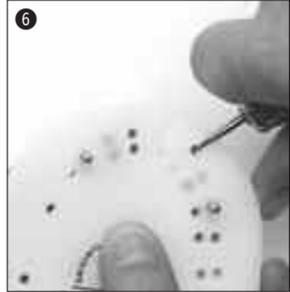
Unfasten the base plate; remove the dies from the mould.

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
Cut a drainage channel in the Giroform base plate to allow refractory die material to escape.

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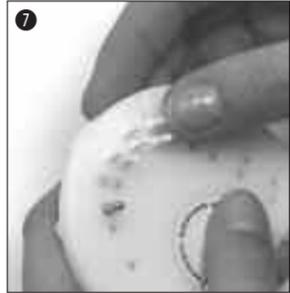Place refractory pins (576480) or tapered double-ended pins (576490) in the base plate.

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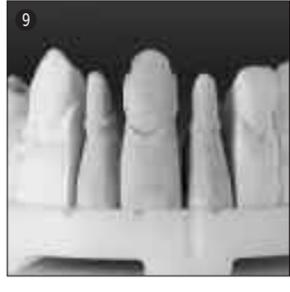
Fit the base plate, close the fasteners, and allow to set according to the manufacturer's instructions.

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Further processing and preparation of the refractory dies should be made in accordance with the porcelain manufacturer's instructions.

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Amann Girschbach AG