VALO[®]

GRAND

ULTRADENT
PRODUCTS, INC.
Improving Oral Health Globally

WARNINGS AND PRECAUTIONS

Read all instructions before operating this unit. The manufacturer accepts no liability for any damage resulting from the improper use of this unit and/or for any purpose other than those covered by these instructions.

- VALO GRAND emits high intensity light very similar to high intensity Quartz halogen lights and must only be used as indicated in this manual.
- DO NOT look directly into the light output. Patient, clinician, and assistants should always wear UV orange eye protection when this device is in use.
- USE CAUTION when treating patients who suffer from adverse photobiological reactions or sensitivities, patients who are undergoing chemotherapy treatment, or patients being treated with photosensitizing medication.
- DO NOT expose soft oral tissues at close proximity. Maintain a safe distance between the lens and the soft tissue. For concern about over-heating soft tissue, consider using multiple short exposures, or use a dual-cure product.
- If using the VALO GRAND LED Curing Light in the Standard and High Power modes and in close proximity of the gingival tissue, DO NOT expose tissue for more than 20 seconds. If longer curing time is required, consider a dual-cure product.
- In Xtra Power Mode, DO NOT expose soft oral tissue for more than 9 seconds. The Xtra Power Mode has a 2 second safety delay to limit oral tissue heating during consecutive curing. If a longer cure is needed, consider a dual-cure product.
- Take extra care to avoid directing light toward soft tissues.
- WARNING: No modification of this equipment is allowed.
- When disposing of electronic waste, (i.e. devices, chargers, batteries and power supplies), follow local waste and recycling guidelines.

Product Information:

Indications for Use: The VALO GRAND is a source of illumination for curing photo-activated dental restorative materials and adhesives.


The VALO GRAND curing light uses a custom, multi-wavelength (395 - 480nm) Light Emitting Diode (LED) for producing the high intensity light capable of polymerizing all light cure dental materials. This intensity will also penetrate porcelain and is capable of curing underlying resin cements similar to a quality halogen light.

The VALO GRAND curing light uses Ultradent VALO rechargeable batteries and battery charger.

Product Components:

- 1 – VALO GRAND curing light
- 4 – Ultradent VALO rechargeable batteries
- 1 – Ultradent VALO battery charger with medical grade 12VDC AC power adapter
- 50 – VALO Barrier Sleeves
- 1 – VALO GRAND Light Shield
- 1 – Handpiece surface mounting bracket with double stick adhesive tape

Overview of Controls:

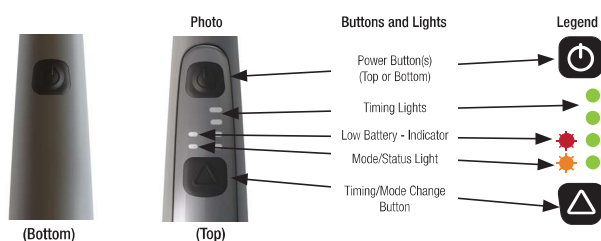Instructions for Use:

- 1- Remove all components from the packaging and examine them. If anything appears to be damaged, return to the manufacturer.
- 2- Charge batteries before using the device (See section Charging and Changing Batteries).
- 3- Insert two fully charged batteries positive (+) end first.
- 4- Replace the battery cover.
- 5- The VALO GRAND handpiece will beep when powering on.
- 6- Select the desired mode (see section Quick Mode Guide).
- 7- Press either power button to activate the light.

NOTE: The VALO GRAND curing light is programmed to cycle from the Standard Power to the High Power to the Xtra Power mode in sequence. For example, to change from the Standard Power mode to the Xtra Power mode, it is necessary to cycle into the High Power mode and then to the Xtra Power mode.

The VALO GRAND curing light stores the most recently used timing interval, and mode, and it will default back to this whenever the modes are changed or if the batteries are removed.

SLEEP MODE: The VALO GRAND curing light will go into Sleep Mode after 60 seconds of inactivity, as indicated by a slow flashing of the mode/status light. Picking up or touching the unit will wake-up VALO GRAND and automatically return it to the last setting used.

WARNING: Storage and Travel: If storing the VALO GRAND for periods longer than 2 weeks, or packing it for travel, always remove the batteries. If batteries are left in the unit for long periods of time without recharging they may become nonfunctional or un-chargeable.

Charging and Changing Batteries

The VALO GRAND comes with 4 rechargeable lithium iron Phosphate batteries.

How to charge batteries

1. Inspect batteries and charger for corrosion, rust, or damage prior to use.
2. Plug charger into electrical outlet.
3. Insert batteries into charger with positive (+) end pointed towards the indicator lights on charger.
4. Green Lights indicate batteries are ready for use.
5. Batteries will take 1-3 hours to charge. Leave batteries in charger until ready for use.

How to change/insert batteries

1. Remove back cap by twisting counterclockwise one quarter turn.
2. Remove batteries.
3. Insert fresh batteries positive (+) side first.
4. Reattach back cap by aligning and gently pushing while twisting clockwise.
The cap will click when fully attached.
5. The unit is ready for use.

Caution

To avoid risk of electrical fire:

1. Keep charger and batteries away from clinical operating to minimize exposure to liquids
2. If cleaning becomes necessary then unplug charger, wet a cloth with isopropyl alcohol, and gently wipe the surface of the charger or batteries.
3. DO NOT spray or wipe electrical or battery contacts.
4. Always prevent liquids from entering any opening.

Note:

- Make sure batteries are inserted into unit positive (+) end first.
- DO NOT insert fingers, instruments, or other objects into the battery compartment.
- DO NOT attempt to clean the gold contacts, batteries or any other part of the battery compartment, Call Ultradent Customer Service if there is a concern.

LOW BATTERIES: The VALO GRAND signals the user that it is time to change the batteries when the Low Battery Indicator Light is flashing red. If the battery charge becomes too low, an audible 3 beep warning sound will occur and the VALO GRAND will not allow further operation until batteries are recharged or new batteries are inserted. (See Warning Light Guide)

Charging Time and Battery Life: Fully charged battery life in the VALO GRAND is dependent on the MODE, TIME interval, battery type, amount of use, and LED efficiency. In general, rechargeable batteries should last 1 - 2 weeks. Non-rechargeable batteries may last 2 - 3 times longer.

- Recommended Recharge Interval: When low battery indicator comes on, or approximately every 1 - 2 weeks, depending on use.
- Extra Batteries: VALO GRAND comes with 4 rechargeable batteries. We also recommend keeping a spare set of non-rechargeable CR123A Batteries on hand in the event of a problem or loss of the rechargeable batteries.
- **Battery Life Expectancy:** Battery life expectancy will vary based on use, handling, and environment. Replace batteries if damaged, dented, or corroded.

Note: If you are moving the light from operatory to operatory and the light does not go into Sleep Mode then the battery life may be affected.

Note:

- If the red light on the charger doesn't turn to green, when the batteries have been charging longer than 3 hours, this means a battery may be degraded (corrupted) and cannot be charged. Try a new battery or call Ultradent Customer Service to order a new set of rechargeable batteries.
- DO NOT use batteries if the printed battery label is torn or removed from the battery. Replace with a new battery and recycle old battery.
- If the light does not function properly, remove and replace batteries and check again.

WARNING - CAUTION

- DO NOT mix rechargeable batteries with non-rechargeable batteries.
- DO NOT attempt to charge non-rechargeable batteries.
- Use only safe rechargeable Lithium Iron Phosphate batteries or non-rechargeable batteries.
- DO NOT store batteries in temperatures over 60°C or in direct sunlight.
- DO NOT autoclave batteries, charger, AC power adapter, or VALO GRAND.
- DO NOT get batteries or charger wet or charge around flammable materials.

If necessary, Ultradent authorizes the following alternative batteries for the VALO GRAND.

Authorized Alternate Batteries
Non-rechargeable, primary batteries 1. Tenergy Propel Photo Lithium: (safe, long lasting, recommended) 2. Titanium Innovations CR123A: (safe, long lasting, recommended) 3. Energizer® 123 4. Duracell® Ultra CR123A 5. SureFire® SF123A 6. Panasonic® CR123A

Installing Hygienic Barrier Sleeves

The hygienic barrier sleeve is custom fitted to the wand and keeps the surface of the VALO GRAND clean. The barrier sleeve helps prevent cross contamination and helps keep dental composite material from adhering to the surface of the lens and wand body, and discoloration and corrosion from cleaning solutions.

Note:

- Using the hygienic barrier sleeve will reduce the light output by 5-10%. Due to the high output power of the VALO GRAND, curing has been shown to be substantially equivalent.
- Barrier sleeves are single patient use only.
- Discard used barrier sleeves in standard waste after each patient.
- DO NOT leave barrier sleeves on the wand for extended periods.
- VALO GRAND must be cleaned and sanitized with appropriate cleaning and/or sanitizing agents after each patient. See section titled **Maintenance and Cleaning**.

VALO Light Shield

The VALO Light Shield is oval-shaped and can be rotated for maximum use. The shield can be used with or without a transparent barrier sleeve.

Cleaning the Light Shield:

Cold disinfect the Light Shield using any surface disinfectant. Do not autoclave.

Precaution: Always wear amber colored safety glasses while the curing light is illuminated.

Quick Mode Guide

Mode Power Level	Standard Power 1000 mW/cm ²				High Power Plus 1600 mW/cm ²				Xtra Power 3200 mW/cm ²
Power Button(s)									
Mode/Timing LEDs									
Time Button Time Options	5s	10s	15s	20s	1s	2s	3s	4s	3s only
To Change Time	Press and release Time Button quickly to cycle through time options.								
To Change Modes	Press and hold the Time Button for 2 seconds and release, VALO GRAND will cycle to next Mode.								
Legend	Solid LEDs: Blinking LEDs:								

Quick Curing Guide: Recommended Curing Times for Optimal Results with VALO GRAND

Exposure times may need to be adjusted due to composite reactivity, shade, distance from the light lens to the composite, and depth of composite layer if it is over 2mm.

Mode	Standard Mode	High Power Plus Mode	Xtra Power Mode
Power Level	1000 mW/cm ²	1600 mW/cm ²	3200 mW/cm ²
Per Layer	One 10 second cure	Two 4 second cures	One 3 second cure
Final Cure	Two 10 second cures	Three 4 second cures	Two 3 second cures

Quick Warning Guide

Power Level Warning	Temperature Warning	Service Warning	LED Warning
Replace batteries	Allow cool down	Call Customer Service for Repair	Call Customer Service for Repair
Low battery: Slow flashing Shut off: 3 beeps, flashing Prohibits operation	3 beeps Slow flashing Prohibits operation	No sound. Flashing, 2 seconds Allows operation	Continuous 3 beeps Single rapid flash Prohibits operations

CURING MODE: Standard Power mode - 1000 mW/cm²

USES: Curing of restorative materials with photo initiators.

TIMING INTERVALS: 5, 10, 15, 20 seconds.

- VALO GRAND defaults to this mode when it is **INITIALLY** powered on. The Status Light is green and the green Timing Lights are solidly illuminated.
- To change timing intervals, quickly press the Time/Mode Button.
- Press either Power Button to cure. To stop curing prior to completion of a timing interval, press either Power Button again.

CURING MODE: High Power Plus mode - 1600 mW/cm²

USES: Curing of restorative materials with photo initiators.

TIMING INTERVALS: 1, 2, 3, 4 seconds.

- From Standard Power mode, press and hold the Time Button for 2 seconds. The green Timing Lights will illuminate and flash. The Status Light will illuminate as a steady orange light, indicating High Power mode.
- To change timing intervals, quickly press the Time/Mode Button.
- Press either Power Button to cure. To stop curing prior to the completion of a timing interval, press either Power Button again.
- To return to Standard Power mode, press and hold the Time/Mode Button for 2 seconds, release, which cycles to Xtra Power Mode, then hold again for 2 seconds, and release to Standard power mode.

CURING MODE: Xtra Power mode - 3200 mW/cm²

USES: The Xtra Power mode is useful for all dental curing. It is especially valuable for deep curing of restorative materials, placing thin veneers, attaching orthodontic brackets, and fast curing in pediatric settings.

TIMING INTERVAL: 3 seconds only (Note: there is a 2 second safety delay to limit heating at the end of each curing cycle. At the end of the delay, beeping indicates unit is ready for continued use).

- From the Standard Power mode, press the Time Button for 2 seconds, release, press again for 2 seconds, and release. Three of the green Timing Lights and the orange Status Light will illuminate and flash, indicating Xtra Power mode.
- Press either Power Button to cure. To stop curing prior to completion of a timing interval, press either Power Button again.
- To return to the Standard Power mode, press and hold the Time Button for 2 seconds.

Notes:

- The VALO GRAND curing light is programmed to cycle from the Standard Power mode to the High Power mode to the Xtra Power mode in sequence. For example, to change from the Standard Power mode to the Xtra Power mode, it is necessary to cycle into the High Power mode and then to the Xtra Power mode.
- If it is necessary to cure near soft tissue, cooling with an air stream is an option for keeping the area cooler.

Duty Cycle: The VALO is designed for short-term operation. At maximum ambient temperature (32°C) 1 minute ON back-to-back cycling, 30 minutes OFF (cooling-off period).

Maintenance and Cleaning

The VALO GRAND curing light is a sealed unit with a hard surface and a scratch resistant glass lens. After each use, moisten a gauze or soft cloth with an approved surface disinfectant and wipe the surface and lens.

Routinely check the lens for cured dental resins. If necessary, use a non-diamond dental instrument to carefully remove any adhered resin.

The barrier sleeve helps prevent cross contamination and helps keep dental composite material from adhering to the surface of the lens and wand body.

Light meters differ greatly and are designed for specific light guide tips and lenses. Ultradent recommends checking VALO GRAND in Standard Power mode. NOTE: the true numeric output will be skewed due to the inaccuracy of common light meters and the custom LED pack VALO GRAND uses.

PRODUCT SAFETY WARNINGS:

- DO NOT autoclave batteries, charger, power adapter, or VALO GRAND.
- DO NOT insert fingers, instruments, or other objects into the battery compartment of the VALO GRAND.
- DO NOT attempt to clean the gold contacts, any part of the battery compartment, batteries or battery charger. Call Ultradent Customer Service if there is a concern.
- DO NOT immerse in any kind of ultrasonic bath, disinfectant, cleaning solutions, or any kind of liquid.
- DO NOT wipe down the VALO GRAND curing light with caustic or abrasive cleaners. See lists of acceptable cleaners below.
- PRECAUTION: Static Electricity – This unit may be susceptible to strong magnetic or static electric fields which could disrupt the programming. If you suspect this has occurred, unplug power supply or remove batteries momentarily.

ACCEPTABLE CLEANERS:

- Lysol Brand III Disinfectant Spray (Recommended)
- Isopropyl alcohol
- Ethyl alcohol-based cleaners
- Lysol® Concentrate (alcohol-based only)
- Cavicide™ products (Non-Bleach)(See warning below)*†

DO NOT USE:


- Strong alkali detergent of any type, including hand soaps and dish soaps
- Bleach-based cleaners (e.g. – Clorox™, Sterilox™)
- Hydrogen Peroxide based cleaners
- Abrasive Cleaners (e.g. – Comet Cleanser™)
- Acetone or hydrocarbon based cleaners
- MEK (Methyl Ethyl Ketone)
- Birex®
- Gluteraldehyde
- Quaternary Ammonium Chloride salt-based cleaners (except Cavicide)

* If used, it may fade the color

† DO NOT use Cavicide1 solution or wipes

Troubleshooting Guide	
If the solutions suggested below do not rectify the problem, please call Ultradent at 800.552.5512. Outside the United States, call your Ultradent distributor or dental dealer. Repairs are only to be performed by authorized service personnel. Ultradent to provide service personnel with documentation to perform repairs. WARNING: When sending units in for repair, service, or calibrations, always remove the batteries from the VALO GRAND and charger. Wrap batteries, charger, adapter, and VALO GRAND separately in the return box.	
Problem	Possible solutions
Light will not turn on	1. Press the Time/Mode or either Power Button to wake from Power Save Mode. 2. Check the red Low Battery Indicator for battery charge status. 3. Check that fresh batteries are correctly inserted into the unit. 4. If red and yellow Warning LEDs are flashing this means the VALO GRAND has reached its internal temperature safety limit. Allow the VALO GRAND to cool down for 10 minutes or use a cool moist towel to cool the unit down quickly. 5. If red Warning LED flashes and beeps continuously, call Ultradent Customer Service for repair.
Light will not turn off	1. Remove batteries and call Ultradent Customer service for repairs.
Light does not stay on for desired time	1. Check that the unit is set to desired Mode. 2. Check the Low Battery Indicator for battery charge status. 3. Check that fresh batteries are properly inserted into the unit.
Light is not curing resins properly	1. Check lens for residual cured resins/composites (see "Maintenance and Cleaning"). 2. Using proper orange UV eye protection, verify the LED die lights are working. 3. Check power level with light meter. 4. Check expiration date on curing resin. 5. Ensure proper technique is being followed accordingly to manufacturer recommendations.
Batteries will not charge	1. Make sure batteries are inserted into the charger in the correct orientation and allow batteries to charge for 1-3 hours. 2. If red lights on the charger do not change to green, call Ultradent Customer Service to order replacement batteries and/or charger. 3. If neither green nor red lights on the charger are visible, call Ultradent Customer Service to order or replace charger and/or AC adapter.
Batteries show corrosion or emits an odor	1. Remove charger from outlet immediately. 2. Call Ultradent Customer Service for new batteries and/or charger.
Charger does not charge batteries	1. Make sure charger is plugged in and AC adapter is plugged into a working power outlet. 2. If green or red lights on the charger are not visible, call Ultradent Customer Service for new charger and/or AC adapter.
Wrapping (plastic covering) comes off of the battery	1. Do not use these batteries in the VALO GRAND. 2. Recycle batteries. 3. Call Ultradent Customer Service to order replacement batteries.

Technical Information	VALO GRAND LED Curing Light
Wavelength range	395nm – 480nm (see qualification below) Effective output Power of VALO GRAND falls within the following wavelength range: • 395nm <= EP <= 480nm Minimal and insignificant power can be found in wavelength ranges: • 380nm – 395nm and 480nm – 510nm

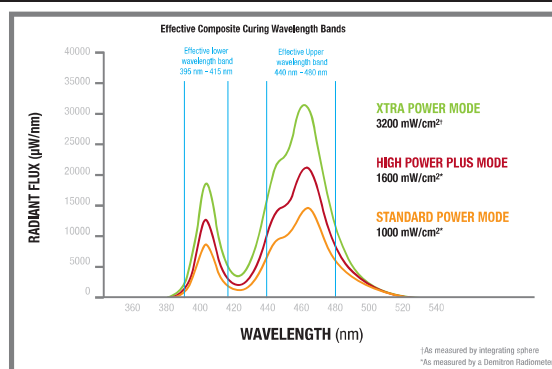Light Intensity	* Standard power – 1000mW/cm² +/-10% * High power Plus – 1600mW/cm² +/-10% † Xtra Power – 3200mW/cm² +/- 20%	* As measured with a Demetron® L.E.D. Radiometer † As measured by the Marc spectrum analyzer Demetron readings are approximate reference power values, but should not be taken as exact values. Dental radiometers will vary from meter to meter, manufacturer to manufacturer.
AC Power Adapter	Globtek Medical Grade with international plug inserts Output: 12VDC, 500mA Input: 100VAC – 240VAC Ratings: Medical Grade Class II, (UL, CE, RoHS, WEEE)	Cord: 6 ft (1.8m), 2.5mm DC connector VALO Charger power supply provides isolation from MAINS power
VALO Charger	VALO 3.6VDC Lithium Iron Phosphate smart battery charger: • Automatic shut off when fully charged • Auto-detection of defective batteries • Charging time: 1 – 3 hours Rating: CE, WEEE	• Protections: Thermal, Overcharge, Short-circuit, reverse polarity - o Red LED – Charging - o Green LED – Empty or Fully Charged - o LED off – short circuit
VALO Batteries	Rechargeable: Safe chemistry Lithium Iron Phosphate (LiFePO₄) RCR123A • Working Voltage: 3.2VDC Ratings: CE, RoHS, WEEE	Alternate use Non-rechargeable: Lithium CR-123A 3VDC (use batteries greater than 1400mAh) Ship batteries in accordance with local regulations

VALO GRAND	Protections: Low Battery, Over Voltage, Over Temperature, LED Failure, Calibration Failure Limitations for Use: VALO GRAND will not allow operation if temperatures exceed 50°C. Ratings: Medical Grade, RoHS, WEEE Weight: • With batteries: 6 oz. (190 grams) • Without batteries: 5 oz. (150 grams) Dimension: (8 x 1.28 x 1.06) inches, (203 x 32.5 x 27) mm, nominal tip diameter 10mm
Operating Conditions	Temperature: +10°C to +32°C (+50°F to +90°F) Relative Humidity: 10% to 95% Ambient Pressure: 700 hPa to 1060 hPa
Storage and Transport Conditions:	Device Storage and Transport: • Temperature: +10°C to +40°C (+50°F to +104°F) • Relative Humidity: 10% to 95% • Ambient Pressure: 500 hPa to 1060 hPa Batteries: • Do not store at temperatures above 40°C/104°F • Do not store at temperatures below freezing 0° C/32° F • Store batteries in a cool, dry and well-ventilated area

Guidance and Manufacture's Declaration for Electromagnetic Emissions

The VALO GRAND is intended for use in the electromagnetic environment specified below. The customer or user should ensure that it is used in such an environment.
WARNING: Use only authorized accessories, cables, and power supplies to prevent improper operation, increased electromagnetic emissions or decreased electromagnetic immunity.

Emissions Test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	VALO GRAND uses batteries and is not affected by EMI, RF, or surge suppression.
RF emission CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	N/A	Harmonic emission and voltage fluctuation testing are not applicable to the VALO GRAND because it is battery powered.
Voltage fluctuations/flicker emissions IEC 61000-3-3	N/A	The VALO GRAND is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings for domestic use.

Guidance and Manufacture's Declaration for Electromagnetic Immunity			
The VALO GRAND is intended for use in the electromagnetic environment specified below. The customer or user should ensure that it is used in such an environment.			
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 15 kV air	± 8 kV contact ± 15 kV air	Physical environment should be restricted to the following: 1. IP Code: IP20 2. Do not immerse in liquid. 3. Do not use around flammable gas. Unit is Non-APG and Non-AP. 4. Storage humidity range: 5% - 95% 5. Storage temperatures range: 10° C - 40° C
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	± 2 kV for power supply lines Note 1: VALO GRAND has no I/O ports	VALO GRAND is battery powered and is not capable of connection to AC MAINS power. Because the VALO GRAND is battery powered, it is not subject to electric transients, surges, voltage dips, shorts, interruptions, or variations on AC MAINS power.
Surge IEC 61000-4-5	± 1 kV line to line ± 2 kV line to earth	± 1 kV line to line ± 2 kV line to earth	
Voltage, dips, shorts, interruptions and variations on the power supply input lines IEC 61000-4-11	<5% U (>95% dip in U for 0,5 cycle) 40% U (60% dip in U for 5 cycles) 70% U (30% dip in U for 25 cycles) <5% U (>95% dip in U for 5 s)	<5% U (>95% dip in U for 0,5 cycle) 40% U (60% dip in U for 5 cycles) 70% U (30% dip in U for 25 cycles) <5% U (>95% dip in U for 5 s) Note 2: Self recovers	The accessory battery charger may be subject to the above but it is separate from and not critical to the operation of the VALO GRAND. If battery voltage of the VALO GRAND drops to 4VDC the unit will not allow operation. The VALO GRAND will turn off. When new batteries are put in and the proper power levels are restored, VALO GRAND will restart and return to the same state before power loss. The VALO GRAND will self-recover in the event of power loss.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical, residential, home health care, commercial, hospital, or military environment.
Note 1: The VALO GRAND is not equipped with any ports or any accessible I/O lines.			

Guidance and Manufacture's Declaration for Electromagnetic Immunity for non-life support systems			
The VALO GRAND is intended for use in the electromagnetic environment specified below. The customer or user should ensure that it is used in such an environment.			
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment guidance
Conduction RF	3 Vrms	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the VALO GRAND, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = \left[\frac{3,5}{V_1} \right] \sqrt{P}$ $d = \left[\frac{3,5}{E_1} \right] \sqrt{P} \text{ 80 MHz to 800 MHz}$ $d = \left[\frac{7}{E_1} \right] \sqrt{P} \text{ 800 MHz to 2.5 GHz}$ P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b Interference may occur in the vicinity of equipment marked with the following symbol: 
IEC 61000-4-6	150 kHz to 80 MHz	150 kHz to 80 MHz	
Radiated RF	3 V/m	3 V/m	
IEC 61000-4-3	80 MHz to 2.5 GHz	80 MHz to 2.5 GHz	
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people. ^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the VALO GRAND is used exceeds the applicable RF compliance level above, the VALO GRAND should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the VALO GRAND. ^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.			

Guidance and Manufacture's Declaration for recommended separation distances between portable and mobile RF communications equipment and the VALO GRAND			
WARNING: Portable RF communications equipment may degrade performance if used closer than 30 cm (12 in"). The VALO GRAND is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The user of the VALO GRAND can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the VALO GRAND as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter (P in watts)	Separation distance according to frequency of transmitter (meters)		
	150 kHz – 80 MHz	80 MHz – 800 MHz	800 MHz – 2.5 GHz
	$d = \left[\frac{3,5}{V_1} \right] \sqrt{P}$	$d = \left[\frac{3,5}{E_1} \right] \sqrt{P}$	$d = \left[\frac{7}{E_1} \right] \sqrt{P}$
0.01	0.12 meters	0.035 meters	0.07 meters
0.1	0.37 meters	0.11 meters	0.22 meters
1	1.7 meters	0.35 meters	0.7 meters
10	3.7 meters	1.11 meters	2.22 meters
100	11.7 meters	3.5 meters	7.0 meters
The VALO Grand has been tested according to IEC 60601-1-2:2014 and passed under radiated field strengths of 10 V/m between 80-MHz to 2,5 GHz. The value of 3Vrms corresponds to V1 and the value 10 V/m corresponds to E1 in the formulas above. For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer. NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

Warranty

Ultradent hereby warrants that this instrument shall, for a period of 2** years from the date of purchase, conform in all material respects to the specifications as set forth in Ultradent's documentation accompanying the product, and be free from any defects in materials and/or workmanship. This warranty applies solely to the original purchaser and is not transferable. This warranty applies solely to the VALO GRAND and does not cover any accessory components such as batteries, chargers, adapters, and adaptive lenses. There are no user serviceable components of the VALO GRAND system. Tampering with VALO GRAND will void its warranty.

The VALO GRAND warranty does not cover customer caused damage. For example; if a VALO GRAND is misused or dropped and the lens breaks, the customer would be responsible to pay for any necessary repairs.

**With sales receipt indicating the date of sale to the dentist.

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U.S. federal law restricts this device to sale by or on the order of a dentist.
For immediate reorder and/or complete descriptions of Ultradent's product line,
refer to Ultradent's catalog or call toll free 1-800-552-5512.
Outside U.S. call +1-801-572-4200 or visit www.ultradent.com.

© Copyright 2017 Ultradent Products, Inc. Patent pending.

Manufactured by
 505 West Ultradent Drive (10200 South)
South Jordan, UT 84095

1003883.5 080717

 **ULTRADENT**
PRODUCTS, INC.