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Bre-Gel BG1

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Instructions for use

Bre-Gel BG1

Before using the product, please read these instructions for use carefully.

The manufacturer will not accept any liability for damage resulting from non-compliance with these instructions for use.

1. Description of the product

The constant quality of Bre-Gel BG1 ensures reusability for a large number of additional models. Excellent flow characteristics and uniform solidification as well as the high stability allow precise reproduction of details of attachments and interlocks. Even deep undercuts can be exactly reproduced thanks to the high elasticity (tensile strength) of more than 30%.

2. Indication

Bre-Gel BG1 is a reversible duplicating gel for the fabrication of duplicate models in CoCr casting.

3. Contra-indication

The duplicate mould made from Bre-Gel BG1 can not be filled (poured) with stone.

4. Processing

4.1.1 Melting in the water bath

Cut Bre-Gel BG1 into small pieces and melt in a jar in the water bath on a heating device (electric hotplate or similar device). During this process, Bre-Gel BG1 must be repeatedly stirred. During heating of Bre-Gel BG1, the temperature must not exceed 93° C. The gel must not be boiled at temperatures exceeding 95° C. Careful melting will increase the service life of Bre-Gel BG1 considerably.

4.1.2 Melting in the automatic duplicating unit

Cut Bre-Gel BG1 into somewhat bigger pieces and fill into the duplicating unit. The heating temperature of the unit must be adjusted to 90° C. The ideal working temperature is between 40° and 45° C. Please adhere to the operating instructions for the respective duplicating unit. After heating and subsequent cooling down to the working temperature Bre-Gel BG1 is ready to use.

4.1.3 Melting in the microwave unit

Cut Bre-Gel BG1 into small pieces and fill into a glass or plastic cup. Place the cup into the microwave unit and melt for 2 minutes at 900 watts. Take out the glass or plastic cup, stir Bre-Gel BG1 and melt again for 1 minute at 600 watts. Immediately after Bre-Gel BG1 starts to bubble, stop the heating process and re-move Bre-Gel BG1 from the microwave unit.

Bre-Gel BG1 must not be boiled (not over 95° C). Careful melting will increase the service life of Bre-Gel BG1 considerably.

4.2 Duplicating

The optimum working temperature is 40° to 45° C. During cooling of Bre-Gel BG1, the material must be stirred frequently to avoid the formation of lumps. Gel time is approx. 40 minutes at air. Then the master model must be devested and Brevest Exakta M, (REF 570 OXM2 O) or Brevest Exakta Speed, (REF 570 OES2 O), is poured into the duplicating mould. Remove the investment material model from the duplicating mould.

5. Reuse and disposal

Clean the duplicating mould under running water. If Bre-Gel BG1 is to be reused, it must be ensured that only clean gel material is heated again. In case of insufficient elasticity or deviations in color or changes in consistency, Bre-Gel BG1 must be disposed of together with regular household garbage.

6. Additional information

These instructions for use are based on state-of-the-art methods and equipment and our own experience. The product may only be used for the indication described under item 2. The user himself is responsible for processing the product. Liability for incorrect results shall be excluded since the manufacturer does not have any influence on further processing. Any occurring claims for damages may only be made up to the value of our products.

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