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# Exaktosil Dubliersilikone

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Additionsvernetzendes Polysiloxan

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## Exaktosil N 15 and N 21

Addition-curing poly siloxane

Before using the product, please read these instructions for use carefully.

The manufacturer will not accept any liability for damage resulting from non-compliance with these instructions for use.

### 1. Description of the product

Exaktosil silicone duplicating materials comprise two components (component A and B) and are addition-linking poly siloxanes. The different shore hardnesses of 15 and 21 Shore A indicate different application areas.

### 2. Indication range

Exaktosil silicone duplicating materials are used to prepare duplicate moulds for duplicate models. Exaktosil N 15 must always be used in combination with a duplicating flask, such as Master-Copy (REF 360 0125 G), whereas Exaktosil N 21 can be used for open duplicating systems, such as the duplicating system according to bredent (REF 520 DBST K, 520 DBST G).

### 3. Contraindication

Based on the current state of knowledge there are no known contraindications.

### 4. Hazard and safety information

4.1 When processing Exaktosil, wear safety goggles, protective gloves and protective clothing.

4.2 Skid risk exists when spilling Exaktosil.

4.3 In case of skin contact, wash with copious amount of cold to lukewarm water and soap.

4.4 In case of eye contact, rinse immediately with water for at least 15 minutes and, if required, consult a doctor.

4.5 When swallowing Exaktosil, consult a doctor.

### 5. Storage and durability

#### Storage:

Exaktosil must be stored in sealed containers at temperatures not above 30 °C.

#### Durability:

After initial use, the material must be processed and used up soon.

### 6. Processing

Prepare the model to be duplicated in the usual way. Stir the components A and B in the mixing ratio of 1:1 to obtain a homogeneous mixture. Homogeneity is achieved when the mass has a uniform color. Then mix under vacuum for 30 seconds. Pour the silicone in a thin, uninterrupted stream into the duplicate mould. Processing time span and setting time will be extended/reduced at higher/lower processing temperatures.

### 7. Technical data

	Exaktosil N 15	Exaktosil N 21
Mixing ratio of components A + B	1:1	1:1
Processing time span from beginning of mixing at 23° C	5 ± minutes	5 ± minutes
Setting time from beginning of mixing at 23° C	30 minutes	30 minutes
Linear dimensional change	0.02 %	0.02 %
Hardness after 1 hour	15 Shore A	21 Shore A
Elongation at rupture	260 %	350 %
Tear resistance	0.72 N/mm <sup>2</sup>	1.25 N/mm <sup>2</sup>
Classification according to DIN EN ISO 14356	Irreversible type 2	Irreversible type 2

### 8. Disposal

Hardened Exaktosil can be disposed of together with household garbage. Exaktosil that has not hardened must be disposed of in accordance with the regulations of local authorities for special waste.

### 9. Symbols

 Lot number

### 10. Additional information

These instructions for use are based on state-of-the-art methods and equipment and our own experience. The product may only be used for the indication described under item 2. The user himself is responsible for processing the product. Liability for incorrect results shall be excluded since the manufacturer does not have any influence on further processing. Any occurring claims for damages may only be made up to the value of our products.

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