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Registrieren | Artikulieren
Registration | Articulation
Enregistrement de l'occlusion
Montage sur articulateur
Registrazione | Messa in
articolatore
Registro | Articulación

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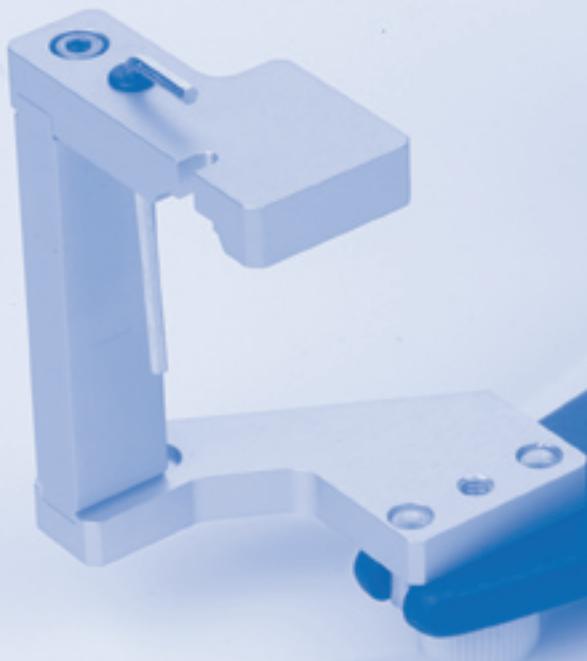
User Manual

Mode d'emploi

Istruzioni d'uso

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
_Deutsch	04
_English	05
_Française	06
_Italiano	07
_Español	08

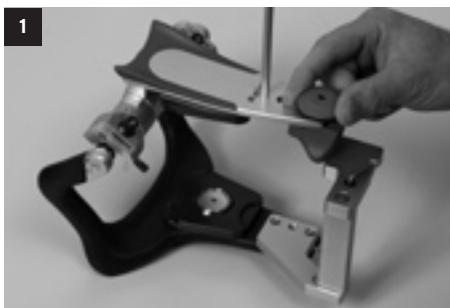AMANN GIRR BACH

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ARTEX® TRANSFER JIG

PRODUCT DESCRIPTION

Another direct alternative to the transfer stand and to the direct facebow transfer. The universal joint with bite fork is attached to the jig holding the registration in axis relation while fixed at the bottom part of the Artex® articulator.

OPERATION/HANDLING

Remove anterior guidance pin and holder. Insert anterior guidance table in top part. Insert transfer jig in bottom part up to stop and secure. The surface of the incisal table lies on the transfer jig. Insert bite fork support in bottom part (via magnet or screwed). Insert 3-D universal joint in the socket on the transfer jig and fix in position with the adapter screw. (**Fig. 1 - 3**)

Turn articulator upside down. Move cross bar of the bite fork support until it contacts the bite fork and tighten up the knurled screw. The cross-bar can be brought closer by means of the screwed thread and/or telescopic fitting. (**Fig. 4**)

Place articulator in working position. Insert maxillary model in the bite tork impressions. Insert base plate in top part. Check space available between model and top part and regulate if necessary, e.g. by trimming the model or applying piaster in two stages. (**Fig. 5 + 6**)

Mix an appropriate quantity of mounting plaster for the space available to a creamy consistency. Moisten the base of the model. Apply mounting plaster to the base plate and the moistened surface of the model. Close top part. Release plaster resistance (relieve stress) by knocking on top part. (**Fig. 7 + 8**)

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Amann Girrbach AG
Amann Girrbach GmbH