# no flame<sup>®</sup> plus

<b>DE</b> Betriebsanleitung	4-6
<b>EN</b> Operating instructions	7-9
<b>FR</b> Mode de fonctionnement	10-12
<b>IT</b> Istruzioni sul funzionamento	13-15
<b>ES</b> Instrucciones de servicio	16-18
EU-Declaration of conformity	20

**AMANNGIRRBACH**

6

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## DESCRIPTION OF THE UNIT

Translation of the original operating instructions

- ❶ Coil opening for instruments
- ❷ LED display for operating status
- ❸ Protective cap
- ❹ ON/OFF switch
- ❺ Mains connection
- ❻ Fan cooler

## DESCRIPTION OF THE PRODUCT

The Noflame®Plus is suitable for most types of waxing up and eliminates the need for conventional Bunsen burners in the laboratory as well as alcohol/gas burners in the dental practice by using the tried and tested induction technique. It heats the instrument just as quickly as a Bunsen burner and heats larger instruments even more quickly. As Noflame®Plus only heats instruments containing steel, it is impossible to burn oneself on the exterior or inside the unit. Another advantage of the induction technique is that there is no carbon residue left on the modelling instrument during heating, which prevents contamination of the modelling wax. A plastic protective cap keeps the coil opening clean. Unlike a Bunsen burner, the Noflame®Plus does not require a gas supply and technicians can continue to use their normal modelling instruments in the usual manner.

## INSTALLATION

Check the unit immediately after removal from the packaging for possible damage caused in transit. If there is any damage, inform the supplier immediately. The unit is supplied ready for operation. Check that the voltage given on the rating plate on the bottom of the unit corresponds to the mains voltage. If you are unsure about the mains voltage, check with an electrician.

To install the unit simply plug in the mains cable at the back (❺) and switch the ON/OFF switch (❹) to ON. The unit is now in stand-by mode. This is indicated by the illuminated green LED display (❷) on the front. The unit does not require the same safety precautions as Bunsen burners as maximum consideration was given to safety aspects during its development. Place the Noflame®Plus in a position comfortable for working.

If multiple devices are used, it is recommended to keep them at minimum distance of 20 inches, otherwise malfunctions may occur.

## OPERATING THE UNIT

Before waxing up place a protective cap (❸) in the spool opening (❶) to catch any drops of wax and keep the unit clean. Hold the modelling instrument in the spool opening (❶) to heat it. Only metallic objects are heated. The heating coil is activated by an integrated sensor that only reacts to instruments containing steel. Activation is indicated by the illuminated red LED display (❷). The instrument is heated just as quickly as with a Bunsen burner and larger instruments are heated even more quickly.

**Caution: A metallic object can become red hot if left for a longer period in the coil opening. Risk of burns! Do not put other metallic objects near the coil as this can interfere with the sensor function!**

The Noflame®Plus has an integrated, quiet-running fan cooler (❻) to protect the heating coil from overheating. The fan cooler automatically switches on if there is an increase in temperature inside the unit.

## SERVICING

Like most Amann Girrbach units, Noflame®Plus is designed to be maintenance-free. Simply ensure that the unit is kept clean. The protective caps have to be used to keep the unit clean and have to be replaced when they become heavily contaminated. Also, the ventilation slits have to not be covered to ensure that the fan cooler functions perfectly.

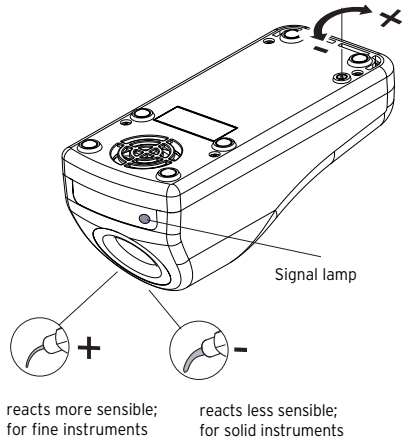
## SENSIBILITY ADJUSTMENT OF RELEASE POINT

The sensibility of the unit is adjusted according to a standard.

However it is possible to tune the unit individually to the instruments / instrumental materials.

The individual tuning has also to be done if the following error occurs immediately after the installation of the unit (without putting an instrument inside the coil opening):

LED display	Cause of error
Red flashes quickly (0,2 second intervals)	Sensor interference

1. Turn the adjusting screw slightly to the left (-).
2. Hold the fine modelling instrument into the coil opening and turn the adjusting screw slightly to the right (+) until the unit responds (signal LED lights up red) and the tool becomes hot!
3. Check the adjustment with a solid instrument.

## ERRORS

The LED display (2) also indicates errors. The following possible errors are indicated:

LED display	Cause of error	Remedy
Red flashes quickly (0,2 second intervals)	Sensor interference (instrument or object containing steel is in the coil opening for more than 6 pulse cycles)	Remove the object from the coil opening and cancel the error by switching the unit OFF/ON (4).
	Sensor defect	Contact customer services!
Red flashes (0,5 second intervals)	Fan cooler blocked	Check the fan cooler and remove any objects blocking the ventilation slits. Cancel by switching the unit OFF/ON (4).
	Fan cooler faulty	Contact customer services!
Red flashes slowly (1 second intervals)	Overheating inside the unit	Switch off the unit and allow it to cool! Switch it on again after a few minutes. If the error persists, contact customer services.

**Technical data:**

Length/Width/Height:	195 x 82 x 85 mm
Weight:	600 g
Power supply:	230V/50Hz
Output:	130 W
Protection:	IP20

**Accessories/Components:**

Order No.:	
116210	Protective caps (10 caps)

**Packing list:**

Order No.:	
116250	<b>Noflame®Plus</b>
	2 Protective caps
	Mains cable
	Instructions for use
	Guarantee card

**SAFETY PRECAUTION**

TheNoflame®Plus was developed for use in the dental sector. The instructions for use should be adhered to during installation and operation. The manufacturer does not accept liability for damages arising from any other use or improper handling. In addition, if the intended application is not listed in the user's manual, the user is obliged to check personally in advance that the unit is suitable for this purpose.

- **Only use Noflame®Plus indoor!**
- **Do not clean the unit with liquids containing solvents!**
- **Keep the ventilation slits clean!**
- **Do not leave the modelling instrument unattended in the coil opening!**
- **Always have repairs carried out by authorised specialists!**
- **Always use original accessories or replacement components!**
- **The electromagnetic radiation of Noflame®Plus at a distance of 30 cm is below the limit values according to ICNIRP.**

Not to be used by anyone fitted with a pacemaker

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**Amann Girrbach AG**