


Technique **without** capping



- 1** Apply bonding agents (e.g. Prelude OneTM) according to IFU.




- 2** Air dry the bonding agent according to IFU.

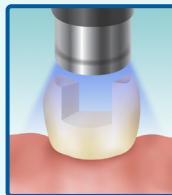

- 3** Light cure the bonding agent according to IFU.



- 4** Fill the cavity with **Bulk EZ**TM, starting at the deepest area.

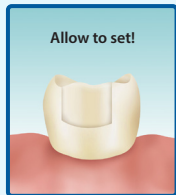

- 5** Allow to set. **Do not light cure** until it is set. From the point of extrusion, time to set is about **90** seconds in oral conditions.



- 6** Light cure for 10 seconds.



- 7** Adjust occlusion and polish.


Technique **with** capping



- 1** Apply bonding agents (e.g. Prelude OneTM) according to IFU.




- 2** Air dry the bonding agent according to IFU.

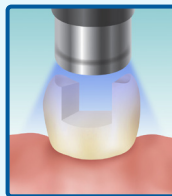

- 3** Light cure the bonding agent according to IFU.




- 4** Fill the cavity with **BulkEZ**TM, starting at the deepest area.

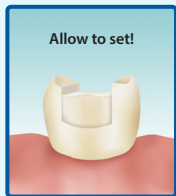

- 5** Allow to set. **Do not light cure** until it is set. From the point of extrusion, time to set is about **90** seconds in oral conditions.



- 6** Cap the top layer with a universal composite (e.g. ZNanoTM).

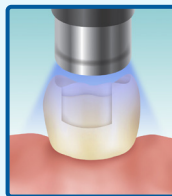

- 7** Light cure the top layer according to IFU.



- 8** Adjust occlusion and polish.