



Fill-Up!™


Dual curing bulk composite

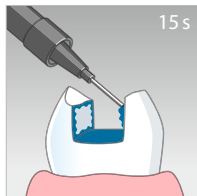


etch

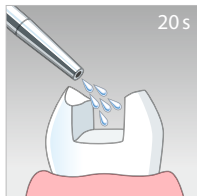


rinse

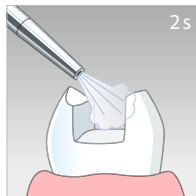


air dry

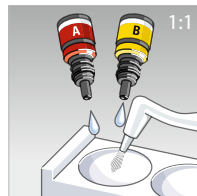


dispense & mix

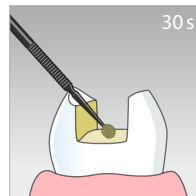


massage

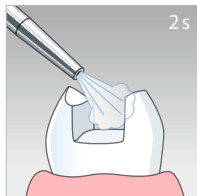


air dry

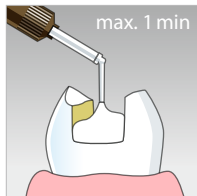


fill

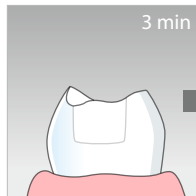

wait for setting

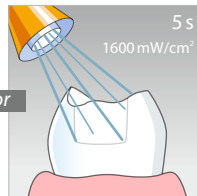

light cure

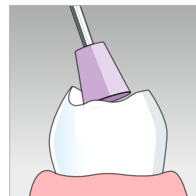

finish

Order Information

Fill-Up!™ Dual curing bulk composite



6001 9341 Fill-Up! Refill

- 1 × 4.5 g Fill-Up! Universal
- 3 × Mixing Tip Short Super Fine
- 4 × Mixing Tip Short Fine

6747 Mixing Tip Short Super Fine

- 40 × Mixing Tip, Ø 1 mm

6759 Mixing Tip Short Fine

- 40 × Mixing Tip, Ø 1.8 mm

6001 9342 Fill-Up! Intro Kit

- 2 × 4.5 g Fill-Up! Universal
- 1 × 2.5 ml Etchant Gel S
- 2 × 3 ml ParaBond Adhesive A & B
- 6 × Mixing Tip Short Super Fine
- 8 × Mixing Tip Short Fine

6001 9355 Fill-Up! Economy Kit

- 5 × 4.5g Fill-Up! Universal
- 15 × Mixing Tip Short Super Fine
- 20 × Mixing Tip Short Fine

Etchant Gel S Ultimate etchant gel



7373 Etchant Gel S Kit

- 2 × 2.5 ml Syringes
- 20 × Application Needles

7876 Application Needles

- 20 × black

ParaBond Chemically curing adhesive system



7493 ParaBond Adhesive A & B Refill

- 1 × 3 ml ParaBond Adhesive A
- 1 × 3 ml ParaBond Adhesive B

