



Important Cleaning and Sterilization Instructions

Composi-Tight 3D™ Soft Face™ rings (orange) and **thin tine G-Rings®** (gray) will maintain their strength and appearance by using the following cleaning and sterilization procedure:

Steam autoclave sterilization only.

DO NOT use chemical vapor sterilizers, alcohol, acetone, ketone, or other harsh chemical disinfectants including ultrasonic cleaning solutions that contain these chemicals.

Sectional Matrix Bands

Matrix bands are single use disposable items. Sterilization prior to use is recommended and can be performed using any common sterilization equipment or chemicals.