

The **ContactEZ IPR Strip System** is an innovative interproximal reduction system that makes interproximal enamel stripping safe and accurate without creating sharp corners or subgingival ledges. Designed for use with Invisalign®, ClearCorrect™, eClinger, Six Month Smiles®, and other orthodontic treatments, these flexible diamond strips curve and conform along the natural contours of the teeth to maximize patient comfort and safety.

- ▶ Ergonomic single-handed design.
- ▶ Provides optimal tactile control.
- ▶ Will not remove excessive enamel.
- ▶ Saves chair-time.
- ▶ Prevent sharp corners or subgingival ledges.
- ▶ Effective for anterior and posterior areas.
- ▶ Central opening for maximum visual perception.
- ▶ Will not cut lips or gums.



ContactEZ IPR Strip System

	Yellow IPR Starter • 0.06mm Single-Sided • Extra-Fine Diamond • Serrated <i>Start to open the contact safely & easily with a serrated edge.</i>	REF 32108
	Red IPR Opener • 0.12mm Double-Sided • Medium Diamond <i>Open the contact and reduce enamel gently on two teeth.</i>	REF 32408
	Dark Blue IPR Widener • 0.15mm Double-Sided • Coarse Diamond <i>Widen interproximal spaces and reduce enamel efficiently.</i>	REF 32508
	Green IPR Extra-Widener • 0.20mm Double-Sided • Coarse Diamond <i>Further widen interproximal spaces safely for ideal alignment.</i>	REF 32608

ContactEZ IPR Optional Strips

	Clear IPR Single-Sided Opener • 0.10mm Single-Sided • Medium-Fine Diamond • Serrated <i>Reduce enamel on the proximal surface of one tooth.</i>	REF 32208
	Cyan IPR Single-Sided Widener • 0.12mm Single-Sided • Medium-Coarse Diamond <i>Further reduce enamel on the proximal surface of one tooth.</i>	REF 32308
	Purple IPR Super-Widener • 0.25mm Double-Sided • Coarse Diamond <i>Further widen the interproximal space between two teeth.</i>	REF 32708
	Brown IPR Mega-Widener • 0.30mm Double-Sided • Coarse Diamond <i>Create an extra-wide interproximal space between two teeth.</i>	REF 32808

Usage Guidelines & Technique

ContactEZ IPR Strips can be best used for small increments of enamel removal (0.1mm increments) in manual reproximation. This technique is especially effective for anterior teeth and for case refinement. ContactEZ IPR Strips are autoclavable (steam heat only, up to 280°F) and reusable, and should be sterilized prior to the first use. The manufacturer recommends to use each strip 3-5 times for maximum efficiency and functionality.

- ① Hold the strip between your thumb and forefinger, insert the strip into the interproximal space, and gently pass the strip back and forth buccolingually until the strip is passive. Do not use vertical force. Start with the thinnest diamond strip, and repeat with thicker strips until you have reduced (stripped) the correct amount of enamel according to your treatment case prescription.
- ② To ensure accurate measurements, polish the proximal surface using the **ContactEZ Gray Final Polishing Strip**.
- ③ Verify the width of the gap created using the **ContactEZ Incremental Thickness Gauge**. The location and amount of any reproximation performed should be recorded in the patient's chart.
- ④ **Pediatric dentists** can use ContactEZ IPR Strips for the serial proximal reduction of primary molars.



Made in the USA • Patented



EMERGO EUROPE
Molenstraat 15
2513 BH, The Hague
The Netherlands



ContactEZ, LLC
217 SE 136th Ave, Suite 105
Vancouver, WA 98684
United States

CONTACTEZ®

217 SE 136th Ave, Suite 105
Vancouver, WA 98684 USA

web www.contactEZ.com
email order@contactEZ.com

tel 360.694.1000
fax 360.694.6191