VOCO

Remin Pro

DE

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GB

## Instructions for use

**Remin Pro** is a water-based cream that contains hydroxyapatite and fluoride. **Remin Pro** is available in the flavours: melon, strawberry and mint. **Remin Pro** provides extra protection for teeth and, in doing so, it helps neutralise acids in acid-forming bacteria in plaque. **Remin Pro** additionally facilitates the neutralisation of other acids in the mouth.

### Indications:

- After tooth whitening
- After professional tooth cleaning
- For the prevention and control of hypersensitivities
- During orthodontic treatment

### Application:

1. Apply a pea-size amount of **Remin Pro** to the teeth with a finger or suitable instrument (toothbrush or swab) and distribute.
2. The patient should distribute the remaining **Remin Pro** in his or her mouth with the tongue. **Remin Pro** and the saliva should remain in the mouth for as long as possible (minimum of 3 minutes) for an optimal result.
3. The patient should spit out the remaining amount. Rinsing should be avoided if possible. The patient should wait at least 30 minutes after the treatment before eating or drinking.

Also for use at home after oral hygiene.

### Storage:

Store **Remin Pro** in a cool, dry area protected from direct sunlight (2°C - 23°C). Do not use the material after the expiry date.

### Precautions:

1. Always close the tube immediately after use.
2. **Remin Pro** contains parabene, fluoride and hydroxyapatite (calcium and phosphate). Do not use in cases of known allergies to any of the ingredients.
3. The application of **Remin Pro** is not recommended for patients under 6 years or for home use on children under 12 years.
4. In case of contact with the eyes, flush thoroughly with plenty of water and seek medical attention.
5. Immediately discontinue treatment and seek medical attention if symptoms of angioedema occur that can be attributed to the ingredients in **Remin Pro**.

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